

105 x 147mm

**Lifemax**

Model **1084**

# Medically Approved Hearing Aid


*Amplifies conversations, radio or TV sounds  
Tone control helps with different types of hearing loss. Suitable for either ear*

## Instruction manual

### Introduction

#### Package Contents:

- Hearing Aid
- 2 Ear buds (L/R)
- 3 Batteries
- Carrying case
- User manual

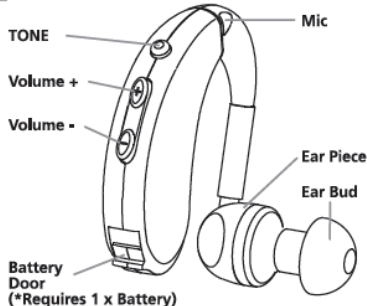

#### SPECIFICATION:

Operation Voltage: 1.4-1.5DCV  
 Max. Current Drain: 6mA  
 T.H.D.: 1KHz(ref.V): 2%  
 Max. Output T.H.D.: 2KHz: 5%  
 SIN Ratio: 54dB  
 Output Power: 0.9mW  
 Frequency Response:  
 Low: 350Hz - 3KHz  
 Medium: 500Hz - 5KHz  
 High: 600Hz - 7KHz  
 Wide: 700Hz - 11KHz

#### OPERATION CRITERIA:

Temperature Range: 5°C - 40°C  
 Humidity: ≤80%  
 Air Pressure Range: 86kPa - 106kPa

### How to use

#### 1. Selecting custom-fit Ear Bud


Included are 2 different size ear buds. Before use, choose from either L or M size to attach to the ear piece as shown in Pic#1. It is important to choose the appropriate ear bud size for maximum comfort & optimal sound quality.

**Caution:** Never insert the Ear Bud alone in your ear, it may be difficult to remove from the ear if it is not attached to the device.

#### 2. Inserting battery

- Remove protective tab from the Zinc - Air battery before use.
- Open the battery door in a direction as shown in Pic#2.
- Insert battery as indicated by the polarity symbols in the compartment as shown in Pic#3.
- Close the battery door.

**Caution:** For optimal performance, we recommended using new battery. When the level of amplification becomes poor or distorted, be sure that the Ear Bud is clear of wax or debris; if the amplification is still distorted, replace with new battery.

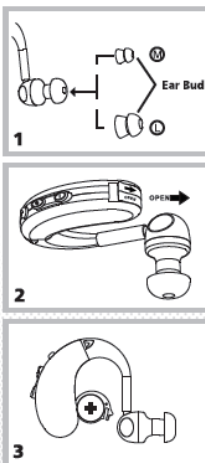

3. Sound volume was preset at level 4 and Tone at "Medium" as shown in Pic#4.
4. Wear the device in your ear by placing the Ear Bud in the ear & positioning the Ear Piece comfortably over the ear. To facilitate insertion of Ear Bud, lightly twist the unit left and right to achieve a snug fit. There is no need to push it into your ear strongly as shown in Pic#5.

**Caution:** If the Ear Bud does not fit your ear and whistling occurs (acoustic feedback), replace it with a different size of Ear Bud.


5. Adjust the volume to the desired listening level (7- level selection: 1 - 7) and press "Tone" button to select among 4 Tones (Medium / High / Wide / Low) according to your personal hearing need as shown in Pic#6.
6. Turn the unit OFF when not in use by adjusting volume to minimum (level 0) or opening the battery door to the lock hinge as shown in Pic#7.

**Caution:** If the unit will not be used for a long period of time, remove the battery from the unit.

#### 7. Cleaning your Hearing Aid

We recommend that you clean your hearing aid regularly (e.g. twice a week) to remove any earwax that may build up as a result of regular use with a cotton tip or brush.

**Caution:** The device is not a water-proof product. Never immerse it completely in water <\*IEC 529 (ECC) level 0\*> It is suggested to place the device back to the carrying case after daily usage and well store it at room temperature with good ventilation and humidity ≤ 80%, and avoid contact with corrosive gases.

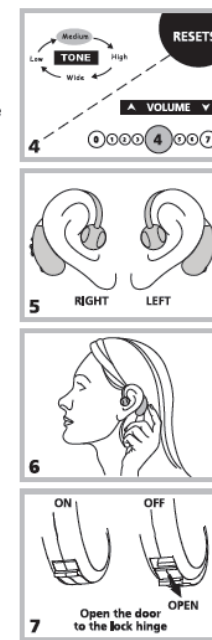

### BATTERY INFORMATION

- This product uses 1 x LR44 or G13A alkaline button cell battery
  - a. Open the battery door as shown in Pic#2.
  - b. Remove the old battery and insert 1 x LR44 or G13A new battery. Observe polarity (+ -) sign when installing battery as shown in Pic#3.
  - c. Close the battery door.
- Use only designated battery type. Do not use re-chargeable batteries. Do not mix battery types.
- Read instructions carefully and install batteries correctly observing the polarity (+ -) sign to avoid leakage.
- Always replace sets of batteries at the same time. Do not mix old and new batteries.
- Do not attempt to recharge non-rechargeable batteries.
- Remove batteries from the product when it will not be used for an extended period of time to avoid leakage.
- Remove exhausted battery to avoid leakage.
- Do not short out battery contacts.
- Do not dispose of batteries in fire.
- Do not immerse electrically operated products in water or other liquid. Wipe clean only.

Please retain these instructions for future reference.

**LIFEMAX LTD.**  
 Unit 1 Pipers Court,  
 Thatcham,  
 Berkshire, RG19 4ER  
[www.lifemaxuk.co.uk](http://www.lifemaxuk.co.uk)

For reasons of health and hygiene, this item is non-returnable.  
 Your statutory rights are unaffected.

**HAPI-030D AF2010-12-01**  
 Foshan Shunde Aiford Electronics Co., Ltd.  
 Xinjiao Industrial Park, Daliang, Shunde,  
 Foshan City, Guangdong Province, China  
 2010-12

EC | REP | Shanghai International Trading Corp, GmbH (Hamburg)  
 Eiffelstrasse 80, 20537 Hamburg, Germany

