

User Instructions

L97792 Economy Shower Chair

General Safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain these instructions in a safe place for future reference.

1.0 Intended Use

The Economy Shower Chair is intended for use in a bathroom or shower to provide a stable seated surface for users with reduced mobility who are unable to stand for long periods. It can be used with or without the back rest.

2.0 Safety Precautions

Do not exceed the maximum user weight, 16 st, 102 kg.

Do not attempt to stand or climb on the chair.

Ensure all four feet are set at equal height to prevent rocking and instability.

Do not use if the rubber feet are missing or damaged.

3.0 Unpacking

Remove packaging and check all parts carefully, refer to contents list. If any parts are missing or appear damaged do not use and contact your supplier immediately.

Contents:

- Bottom Legs X 2
- Countersunk Bolts X 4
- Cupped Washer X 5
- Seat X 1
- Round Head Bolts X 5
- Back Support Tube X 2
- Back Rest X 1

4.0 Assembly

Put the seat upside down on the floor.

Align the legs on the seat such that they sit in the grooves on the plastic moulding, see figure 2.

Maximum User Weight



Figure 1 – L97792 Shower Chair

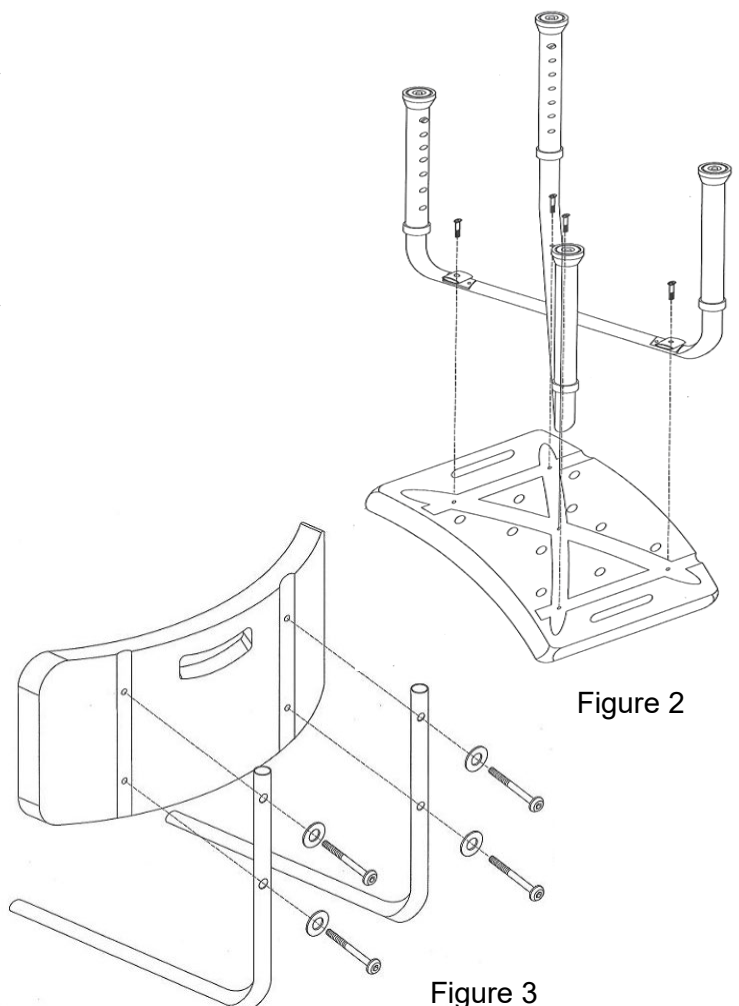


Figure 2

Figure 3

Secure in position by inserting a round headed bolt through a cupped washer and into the central fixing point on both the legs and seat.

Fit the four countersunk bolts through the legs into the seat moulding.

4.1 Optional Back Rest

Align the back support tubes such that they sit in the grooves on the back of the back rest moulding.

Secure in position by inserting the four round head bolts through cupped washers and into the back support tubes and back rest, see figure 3.

Fit the backrest to the seat by slotting the back rest tubes into the locating tubes on the underside of the seat, see figure 4.

Depress the button clip to allow the tubes to slide, ensure it clicks into place to secure.

Please Note: Two back rest positions are available dependent upon the comfort of the user. Ensure the same button clip is used on both tubes to prevent the backrest being angled.

4.2 Height Adjustment – Legs only

Press the button clip and slide the leg extension to the desired position.

Allow the button clip to click into the nearest adjustment hole.

Repeat for all four legs.

Use the markings on the legs to ensure that all four legs are set to equal height to prevent rocking and instability.

5.0 Cleaning

Clean regularly using a general purpose neutral detergent and warm water.

Check the suitability of cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.

Allow the product to dry fully prior to use or storage.

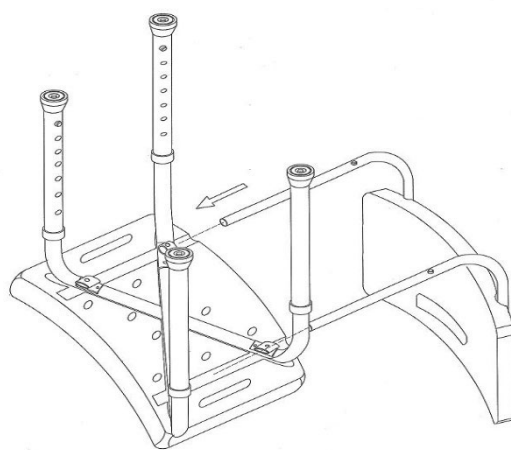


Figure 4

6.0 Maintenance

Check that all fixings are inserted correctly and that there is no excessive movement.

Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the rubber feet.

Immediately report any suspected faults or damage to the supplier and take the product out of use.

7.0 Product Disposal

Clean the product thoroughly before disposal.

Product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through local Trade Waste Collectors.

8.0 Product Guarantee

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS Healthcare

Sherwood House, Cartwright Way, Forest Business park, Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK

Tel. Customer Services: +44 (0)345 121 8111

Email: customerservices@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.

Please visit one of our websites to view our full range of products:

www.healthcarepro.co.uk or www.completecareshop.co.uk