

# USER INSTRUCTIONS

## NRS WHEELED COMMODE

L22056	Wheeled Commode Fixed Height	M66119	Wheeled Commode Height Adjustable
N54411	Economy Wheeled Commode Fixed Height	S30063	Wheeled Commode Powder Coated

### ⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

### Maximum User Weight



## 1. INTENDED USE

The Wheeled Commode range has been designed for use as a traditional commode or for use over a standard toilet. Subject to completion of a suitable risk assessment, the wheeled commode can be used for short transfers to move the user over the toilet. It features removable arms to enable side transfers.



## 2. SAFETY

**Warning:** Do not exceed the maximum user weight specified for each product.

Always perform a risk assessment prior to use, including safe moving distance.

Do not use as an alternative to a wheelchair.

Do not use in a shower or as a shower chair.

Do not use the product outdoors.

Only use indoors on level surfaces and take care over door thresholds.

Always engage the brakes and swing away or lift off the footrests when the user is transferring onto or off the commode.

Take care to ensure all body parts are inside the perimeter of the commode when passing through doorways.

## 3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

### 3.1 Contents:

Part	L22056 N54411	M66119 S30063	Qty
Main Chassis & Castors	✓		1
Main Chassis		✓	1
Seat Cover	✓	✓	1
Detachable Arms	✓	✓	2
Detachable Back	✓	✓	1
Potty	✓	✓	1
Footrest	✓	✓	2
Adjustable-Height Castors		✓	4
Hand Wheel	✓	✓	2
E-Clip		✓	4

## 4. ASSEMBLY

### 4.1 Fitting the Castors (M66119 and S30063):




Slide the castors into the main chassis, set to the desired height, then secure with the e-clips.

**Note:** Make sure that all castor legs have been set to the same height.

**Note:** The height should be set for user comfort but must allow for adequate clearance if being used over a toilet.

#### 4.2 Fitting the Armrests:

**Note:** The armrests are left and right-handed, please check the labels on the armrest. The padded overhang should face the outside of the chair.

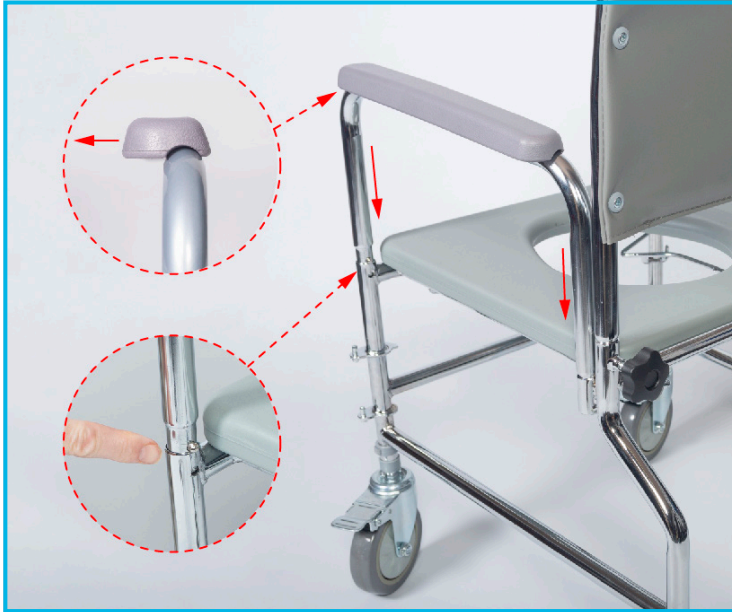

Attach the arms by slotting the tube inside the main chassis frame on either side of the seat moulding. Push and hold the button-clip, then push the armrest down until the button-clip clicks into place. Repeat the steps for the armrest on the opposite side.

**Note:** Make sure the button-clip has fully engaged the hole and that the armrest is fully secure.

#### 4.3 Fitting the Backrest:



2. Slot the backrest into the main chassis frame, then turn the hand-wheels clockwise to secure.

**Note:** Make sure the backrest is fully secure before moving to the next step.

#### 4.4 Fitting the Footrests:



Slot the footrests onto the studs on the main chassis frame.



Rotate the footrest into the working position and secure with the locking clip.

**Note:** Make sure the locking clip has clicked into place and has fully secured the footrests.



Unfold the footrests into the working position.

#### 4.5 Fitting the Commode Pan:



Slide the commode pan onto the metal runners at the rear of the commode.

**Note:** Make sure that the commode pan is fully secure before use.

#### 4.6 Fitting the Padded Seat Cover



Fit the padded seat cover over the moulded seat.

### 5. IN USE


#### 5.1 Adjusting the leg height (M66119 and S30063):



Remove the e-clip and slide the castor leg to the desired position, then replace the e-clip to secure. Repeat for the other castor legs.

**Note:** Make sure that all castor legs have been set to the same height.

#### 5.2 Removing the Footrests:

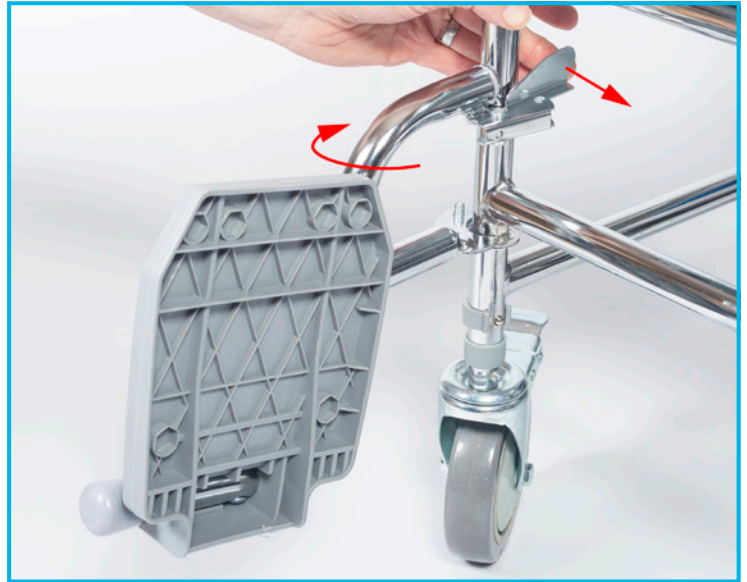

Push the fastener clip to release the footrest. Twist the footrest outwards and lift off from the main chassis frame studs.

#### 5.3 Adjusting the Footrest Height.



Use the spanner provided to loosen the locking nut on the end of the tube.

Slide the inner tube to the desired height.

Use the spanner to re-tighten the nut and secure the position.

#### 5.4 Operating the Brakes



Push the lever down to lock the brakes.



Flip the lever up to unlock the brakes.

## 6. CLEANING INSTRUCTIONS

### 6.1 Cleaning the Commode Pan:

Remove the commode pan by carefully sliding it along the metal runners at the rear of the commode. Place the lid over the commode pan to prevent spillages and splashing.

Dispose of the contents into a proper sluice or toilet.

**Warning:** *Never empty the contents into an open drain, sink or basin as this could cause a severe health risk.*

Wash out any residue with a disinfectant solution and rinse thoroughly with fresh water.

Refit the commode pan to the commode, refer to Section 4.5.

### 6.2 Cleaning the Commode Chair & Frame:

Thoroughly wash the commode after each use using a general purpose neutral detergent and warm water.

To disinfect, in particular the seat, wipe with a dilute bleach solution and rinse thoroughly with fresh water.

Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.

## 7. MAINTENANCE

Inspect the product regularly for signs of wear or damage. Pay particular attention to the upholstery.

Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

Periodically spray a water dispersing agent (e.g. WD40) onto all joints and castors.

Check that all components are correctly fitted and secure. If necessary tighten the nuts, screws and hand wheels.

Check the castors are free to rotate, remove any build-up of dirt/debris.

Frequently check the brakes are operational and there are no signs of wear.

Store the commode on a flat surface, with the brakes locked, out of reach of children.

## 8. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal, plastic and PVC and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

## 9. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

**NRS HEALTHCARE** Sherwood House, Cartwright Way, Forest Business Park  
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: +44 (0)345 121 8111

Email: [customerservice@nrshealthcare.co.uk](mailto:customerservice@nrshealthcare.co.uk)

**NRS Healthcare** is a leading supplier of community equipment and daily living aids.  
Please visit one of our websites to view our full range of products:  
[www.healthcarepro.co.uk](http://www.healthcarepro.co.uk) or [www.completecareshop.co.uk](http://www.completecareshop.co.uk)