





N72140 - Seahorse Plus Chest Harness Medium

Fitting instructions and instructions for use



Fitting and using the Chest Harness

The chest harness is supplied as an accessory. It should be fitted to the chair before use. Please follow the instructions below:

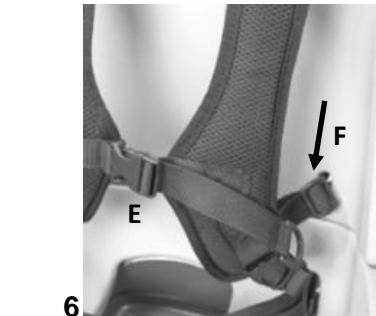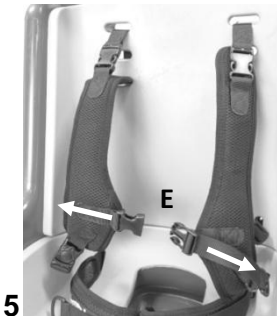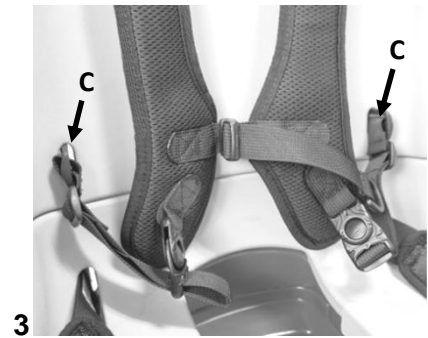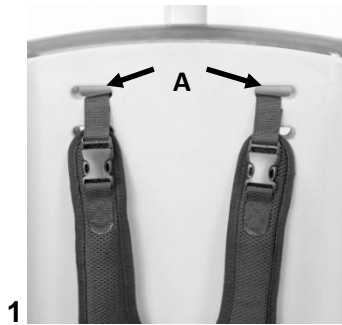

1. The chest harness is fitted by feeding the top webbing straps (A) through the upper slots in the chair seat / back liner (where fitted). There are three pairs of slots, allowing a different height setting for the harness
2. The top webbing straps should then be fed down through the cam-lock (B) and the cam-lock closed. Leave plenty of slack at this stage
3. The buckles on the lower straps (C) should be fed through the lower slots in the chair seat / back liner (where fitted), by rotating through 90 degrees and pushing firmly
4. The lower straps should then be pulled tight at the front, with the buckles (D) flush at the rear of the seat. Make sure the buckles cannot pull through the seat, by tugging firmly
5. Open the central buckle on the harness and loosen the front strap in the buckle, and sit the user in the chair. Tighten the upper straps from the rear of the chair, by opening the cam lock (B) and pulling down on the strap before closing the cam lock
6. To adjust the bottom straps, adjustment sliders (F) can be tightened on either side. Close the central buckle (E) to secure the user

Issue 2 17.06.16