

# USER INSTRUCTIONS

## SECURE PATIENT TURNER

L84438

Secure Patient Turner

### ⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

#### 1. INTENDED USE

To rotate a standing patient during a transfer from e.g. wheelchair to a chair.



#### 2. SAFETY

**Warning:** The maximum user weight must not be more than 190 kg (30 st).

Please note the user must not be transported using the device.

Ensure the handle is facing the wheel and that the brake is locked before manoeuvring the Secure Turn. Never assist a user to alight from the wheeled side of the Secure Turn.

Transfers should only be carried out to the left or right through 45° or 90°.

User alighting should only be performed when the brake is engaged and the Secure Turn is stabilised by a carer.

The Secure Turn should be inspected on a regular basis for signs of damage or wear that might affect safe use.

#### Maximum User Weight



#### 3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

#### 4. IN USE

1. Familiarise yourself with the operation of the Secure Turn before attempting to transfer a user, in particular the brake locking mechanism.
2. To manoeuvre the Secure Turn into position, with the handle facing the wheels and brake locked in position, tilt the device back onto the wheels and move into position.
3. Place the seat/chair that the user is transferring to into position at 45° or 90° to the user's current position.



4. Place the turntable up to the user's feet with the wheels facing the carer.
5. The user should position their feet, evenly spaced, on the turntable mat. The user may need to move to the edge of the seat before rising to reduce the effort required.



6. With the Carer stabilising the Secure Turn, the user raises themselves into a standing position.
7. When the user is standing and stable (holding on to one of the pairs of handles on the frame), the carer releases the brake and rotates the turntable to the required position allowing the brake to lock into position.
8. The carer stabilises the Secure Turn whilst the user lowers themselves onto the seat/chair.
9. After the user has removed their feet from the turntable, manoeuvre the Secure Turn away from the user.

## 5. MAINTENANCE

Inspect the product weekly for any signs of wear or damage.

Check the rotating platform can swivel freely.

Any suspected faults or damage should be reported to the supplier immediately and the product taken out of use.

## 6. CLEANING INSTRUCTIONS.

Clean regularly using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.

Do not clean this device using an industrial cleaner or high-pressure hose as this may affect the smooth operation of the turntable.

## 7. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

## 8. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

If you have any problems with our products, please contact us at the address below.

If you have experienced an injury, contact us but feel free to contact the Medicines and Healthcare products Regulatory Agency at:  
[devices.regulatory@mhra.gov.uk](mailto:devices.regulatory@mhra.gov.uk)



**NRS HEALTHCARE** Sherwood House,  
 Cartwright Way, Forest Business Park, Bardon  
 Hill, Coalville, Leicestershire, LE67 1UB, UK.



**NRS HEALTHCARE** 4C Altona Road, Blaris  
 Industrial Estate, Lisburn, Co Antrim,  
 Northern Ireland BT27 5QB

Tel: Customer Services: +44 (0)345 121 8111

Email: [customerservice@nrshealthcare.co.uk](mailto:customerservice@nrshealthcare.co.uk)

**NRS Healthcare** is a leading supplier of community equipment and daily living aids.

Please visit one of our websites to view our full range of products:

[www.healthcarepro.co.uk](http://www.healthcarepro.co.uk) or [www.completecareshop.co.uk](http://www.completecareshop.co.uk)