

USER INSTRUCTIONS

EASYFIT PLUS BED RAIL MK2

P63991

Standard

P64004

Tall

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Failure to do so can result in injury. Keep these instructions in a safe place for future reference.

INTENDED USE

The EasyFit Plus Bed Rail MK2 provides stable support to aid user transfer in and out of bed, moving from a lying to a sitting position as well as assist user repositioning.

Designed for use with most types of domestic beds including; divans with feet / castors, framed slatted beds and electric profiling beds.



Figure 1: EasyFit Plus Bed Rail MK2 (Tall)

SAFETY

WARNING To avoid possible head entrapment, position the bed rail at least 320mm (12.5") away from both headboard and footboard.

DO NOT exceed product Maximum User weight: 160Kg (25st).

ALWAYS ensure that the mattress is as close as possible to the bed rail. Any gap could allow the client to become wedged in-between.

DO NOT use on electrically height adjustable beds unless the height adjustment function has been disabled, as this could damage the bed rail and / or the bed frame when operated.

ALWAYS ensure the bed rail is firmly clamped to the bed frame. Check regularly to ensure the bed rail is securely clamped.

Maximum User Weight



ALWAYS detach the handrail from the base before carrying or storing the product.

NOT suitable for use by children.

UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.


Contents:

1. Base Frame x 1
2. Handrail Assembly x 1

Recommended Bed Rail Heights

P64004 (Tall) for
beds heights more
than 600mm (24")


P63991 (Standard)
for beds heights less
than 600mm (24")

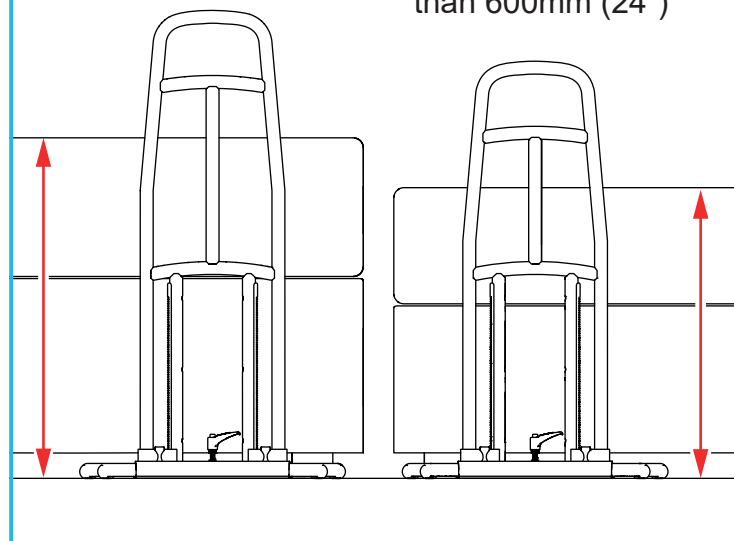

NRS recommends fitting the standard bed rail (P63991) on beds with a mattress height up to 600mm; for higher beds, the tall bed rail (P64004) should be used.

HANDRAIL ASSEMBLY

1. Fit the Handrail Assembly to the Base:

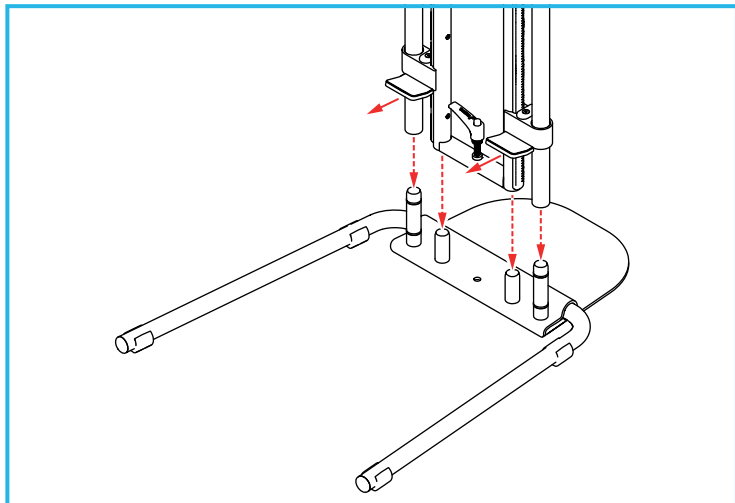


Note: Make sure the bed clamp plates point in the direction shown.

Slot the handrail assembly onto the 4 base pins. Push the handrail down until it makes contact with the base.


PREPARE BED RAIL FOR INSTALLATION

1. Lower the Ratchet Clamps

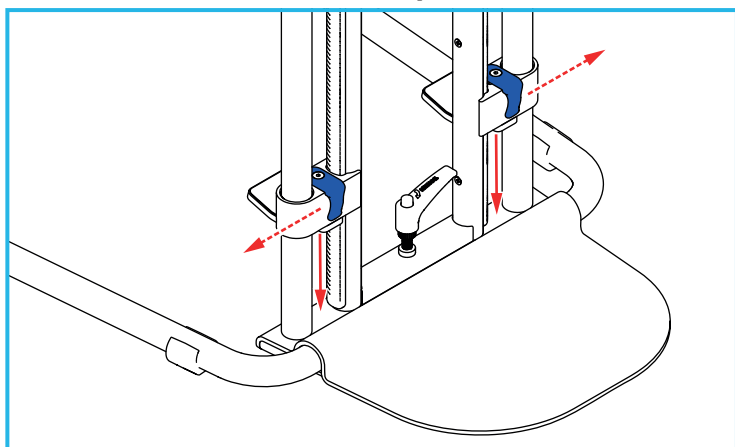

Slide the blue clamp levers outwards to unlock the ratchet clamps. While holding the clamp levers open, slide the ratchet clamps down until they make contact with the base.

2. Lower the Tightening Lever

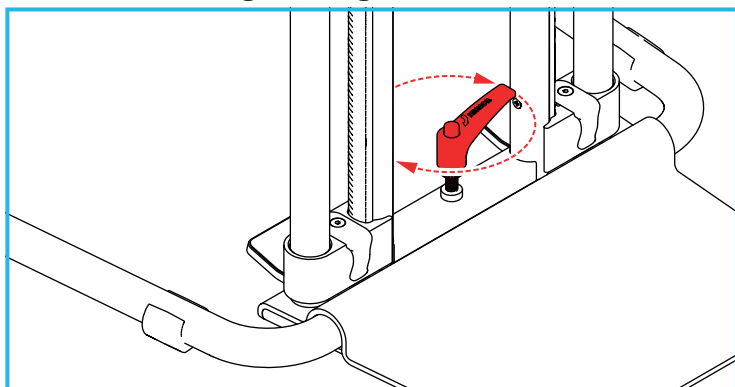


Turn the tightening lever clockwise until you start to feel slight tension. Do not over tighten as this will start to lift the handrail assembly off the base frame.

FITTING TO THE BED

Make sure that the floor surface is level and even.


The bed base must be at least 30mm above the floor and clear of obstructions.

1. Slide the Bed Rail Under the Bed

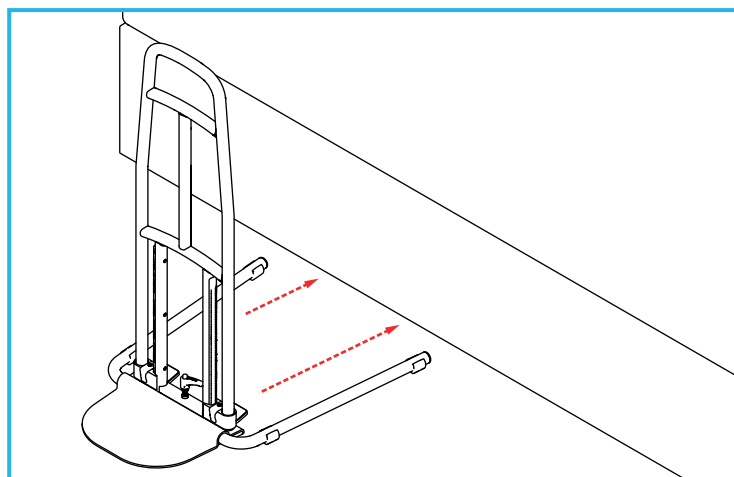

Slide the bed rail under the bed, in an approximate position, until the handrail contacts the bed.


2. Position the Bed Rail

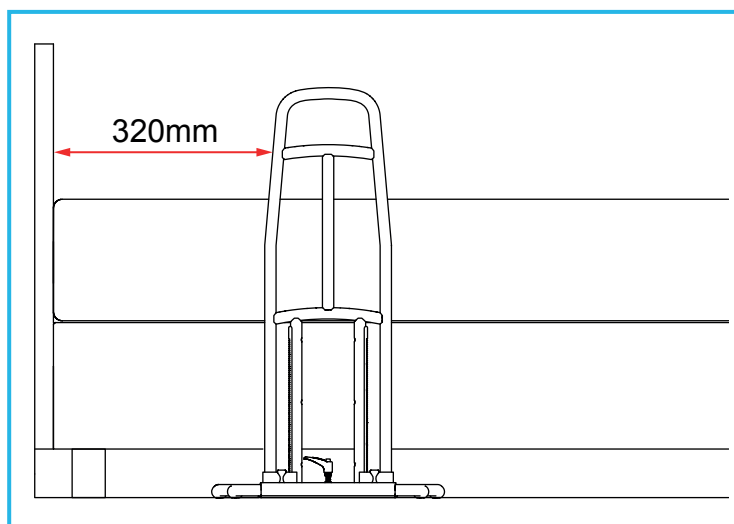


Warning: The bed rail position must be a minimum of 320mm from the bed headboard/wall.

Position the bed rail so the user can reach the handrail from a laying position without having to stretch.


3. Clamp the Bed Rail to the Bed

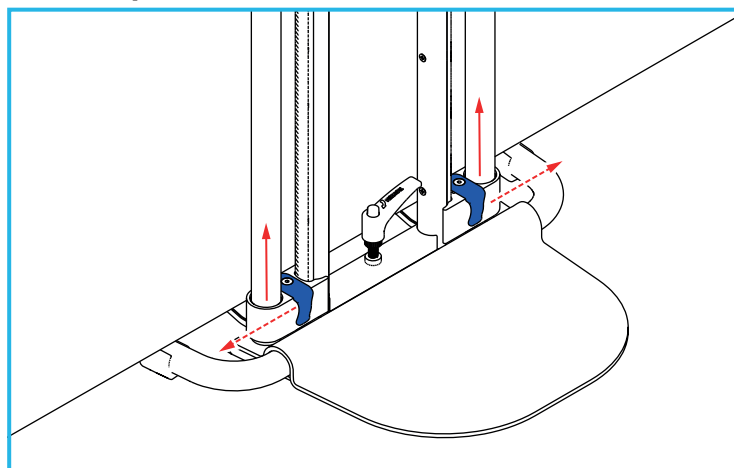

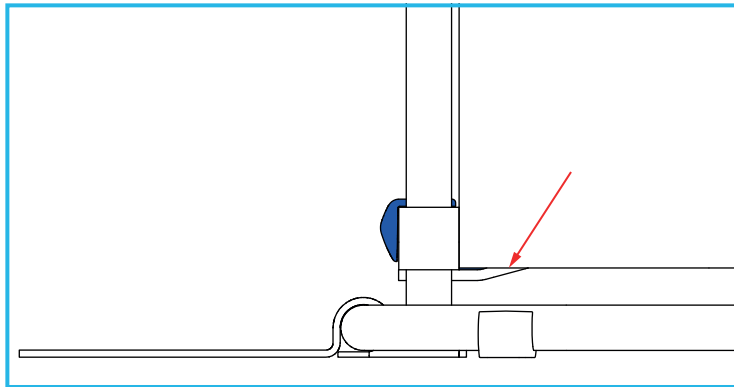

Slide the blue clamp levers outwards to unlock the ratchet clamps. While holding the clamp levers open, slide the ratchet clamps up until the clamp plates make contact with the bottom of the bed frame.

4. Lock the Clamps

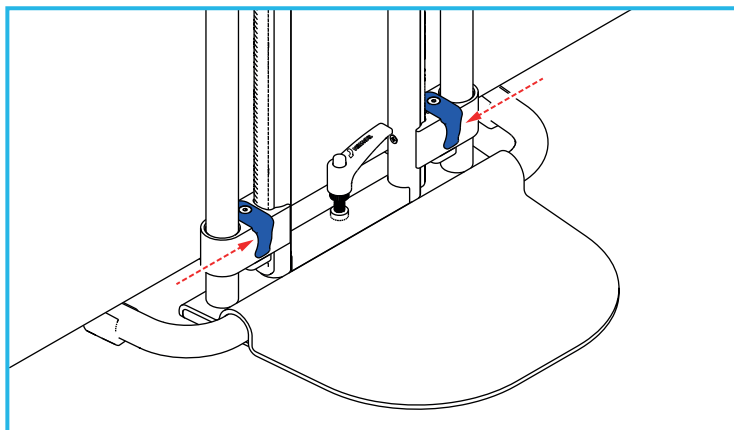


Release the blue ratchet clamp lever to lock the clamps in place.

USING THE TIGHTENING LEVER

Before use, the bed rail must be fully secured using the tightening lever. The tightening lever operation is different depending on the type of bed you have.

1. Slatted Bed Frame Tightening

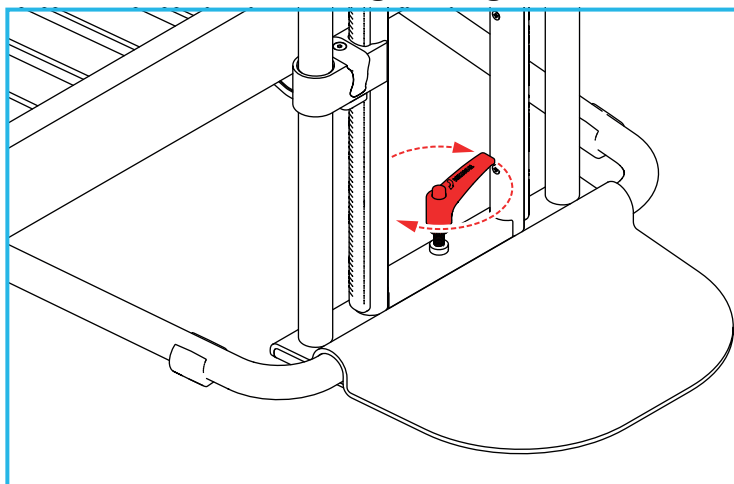


Turn the tightening lever clockwise until you start to feel tension and the bed rail feels secure.

Note: Lightweight beds may raise off the floor when tightening the tightening lever. If this happens, slacken the lever until all four corners of the bed are in contact with the ground.

2. Divan and Low Bed Frame Tightening


Full turns of the tightening lever may not be possible on some divan and low frame beds. Use the following steps to adjust the bed rail;

Step 1

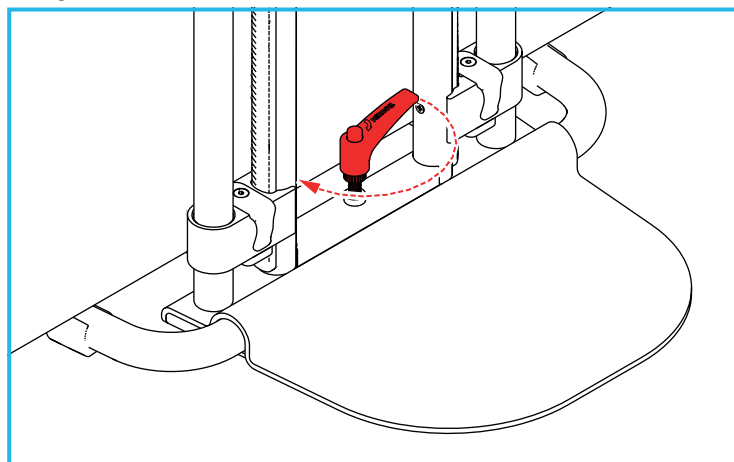

Rotate the tightening lever clockwise by half a turn.


Step 2

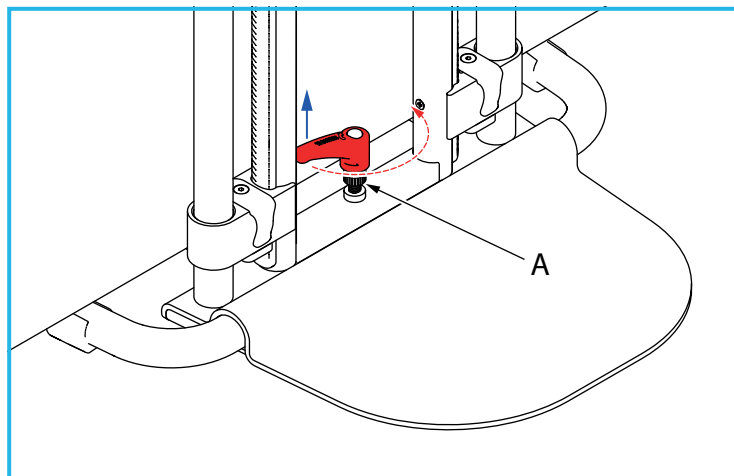

Lift the tightening lever to disengage it from the thread. While holding the lever up, rotate it anti-clockwise by half a turn.

Note: You may need to hold the threaded part (A) to stop it rotating with the lever.

Step 3

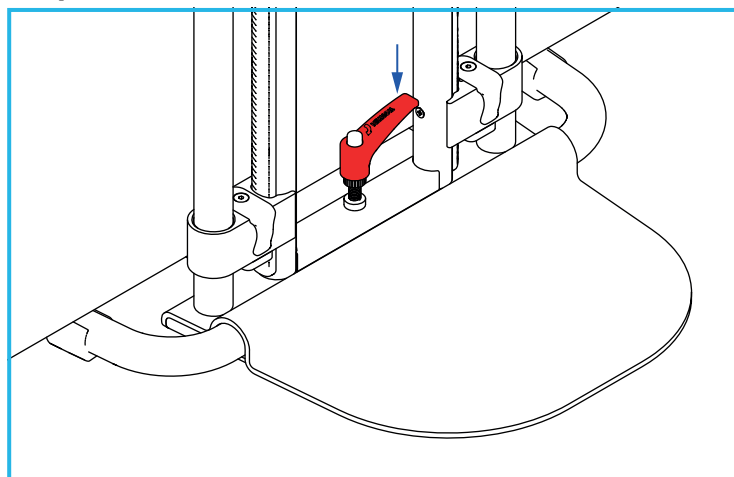

Release the tightening lever, this will re-engage the thread.

Repeat steps 1, 2 and 3 until you start to feel tension and the bed rail feels secure.

It is recommended that the bed rail is checked at monthly intervals, to make sure it is securely fixed to the bed.

If the bed rail has become loose, it must be re-secured using the correct tightening lever method for your bed type.

Note: *If the bed rail cannot be re-tightened sufficiently, it must be removed and reinstalled.*

REMOVING THE BED RAIL

The bed rail can be removed by turning the tightening lever anti-clockwise a few turns until the bottom of the handrail touches the base plate. Then release the 2 ratchet clamps.

CLEANING

Clean regularly using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water.

This product is suitable for thermal decontamination and can be steam cleaned or autoclaved on cleaning cycles up to 80°C.

Allow product to fully dry prior to use/storage.

MAINTENANCE

Inspect the product at regular intervals and before re-use for signs of general wear or damage.

Any suspected faults or damage should be reported to the supplier immediately and the product taken out of use.

ACCESSORIES & SPARE PARTS

P64016: Spare Ratchet Clamp Kit

1. LH and RH Ratchet Clamp Assembly (Pair)
2. Ratchet Rack x 2
3. Ratchet Rack Screws x 6

P11863: Spare Parts Kit

1. Rubber Pads for Clamp Assembly x 4
2. Base Feet x 4
3. Base Tube End Caps x 2
4. Tightening lever x 1
5. Base Pin O'rings x 4

For more details please visit our website:

www.nrshealthcare.co.uk or call Customer Services.

PRODUCT DISPOSAL

This product must be cleaned thoroughly before disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.



NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel. Customer Services: 0345 121 8111

Email: customerservices@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids, to view our full range, visit our website:

www.nrshealthcare.co.uk

P63991-UI-1