

USER INSTRUCTIONS

NRS LAP-STRAP

N99089	NRS Lap-Strap Small	N99091	NRS Lap-Strap Medium
N99108	NRS Lap-Strap Large		

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.




1. INTENDED USE

Available in small, medium or large, the NRS Lap-Strap has been designed for use as a positioning aid for children and young adults. The Lap-Strap can be used with a variety of NRS products including: the Seahorse Plus (N72012/N72024/N72036) and NUVO Children's Commode (P77916).

2. LAP STRAP SIZE GUIDE

Please use the following information as a size guide to select the correct size lap strap for the user.

Lap Strap	Overall Length (A)	Padded Section (B)	Padded Section (C)	Padded Section Total Length (D)	Optimal Age Range
Small (N99089)	400-800mm	245mm	245mm	490mm	5-9 yrs
Medium (N99091)	500-1000mm	275mm	275mm	550mm	10-14 yrs
Large (N99108)	550-1200mm	305mm	305mm	610mm	15+ yrs

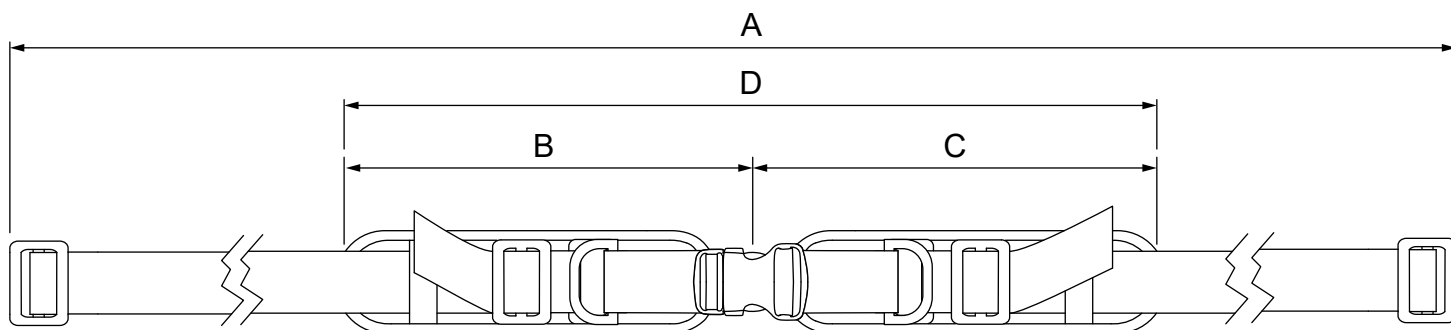

3. SAFETY

Warning: Only use the lap-strap recommended for your product.

Warning: Make sure you have the correct size lap-strap for the product, refer to the guide in Section 2.

Warning: Make sure the lap-strap is correctly tightened for the user every time the user is seated. Incorrectly tightened lap-straps can result in the user sliding forwards.

Warning: Check regularly for signs of wear or damage and replace where necessary.


4. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.


5. FITTING THE LAP-STRAP

Please refer to the user instruction provided with the product you are fitting the lap strap to. The lap-strap must be fitted to the product before use.


6. USING AND ADJUSTING THE LAP-STRAP

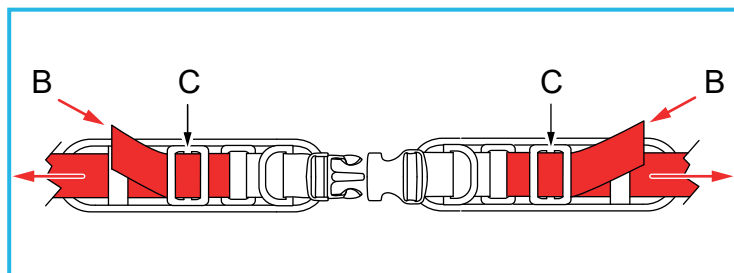


1. Loosen the lap-strap on each side by feeding the loose end of the webbing (B) through the slider (C) until the lap-strap is at its longest setting.

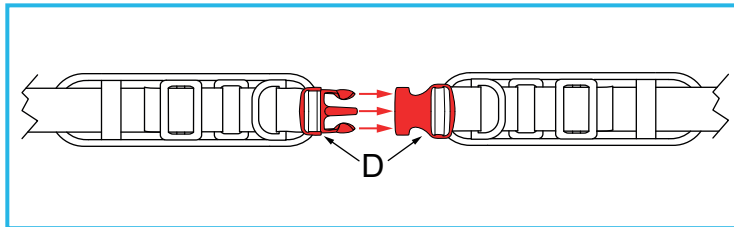

- Position the user in the chair, then fasten the lap-strap, using the buckle (D), loosely around the users waist.

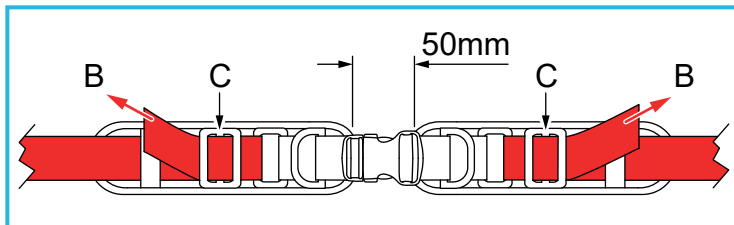

- Adjust the fit of the lap-strap by feeding the excess webbing (B) through the sliders (C) on each side until the lap-strap is a loose fit. Make sure there is a gap of approx 50mm (2") between the padded sections.

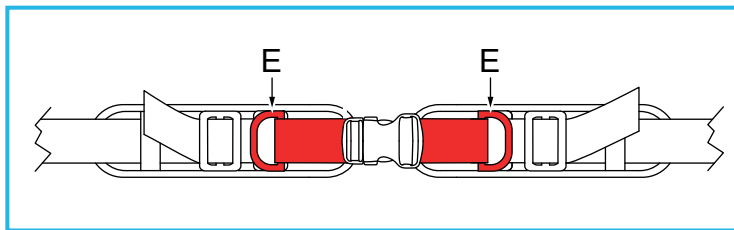

- For final adjustment to ensure a tight fit, pull on the two D-shaped toggles at the front (E) to close the gap between the padded sections.

7. MAINTENANCE

Check all straps and harnesses for any signs of damage to the webbing or to the buckles. Damaged items must be replaced immediately.

Please contact NRS Healthcare Customer Services to report a fault or book an inspection by a service technician.

8. CLEANING INSTRUCTIONS.

- Clean with sponge and warm soapy water.
- Do not use bleach or solvents.
- Do not steam clean, tumble dry or dry clean.
- Allow product to fully dry prior to use/storage.

9. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from plastic and fabric and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

10. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; fair wear & tear, abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

If you have any problems with our products, please contact us at the address below.

If you have experienced an injury, contact us but feel free to contact the Medical and Healthcare products Regulatory Agency at:

devices.regulatory@mhra.gov.uk



NRS HEALTHCARE Sherwood House,
Cartwright Way, Forest Business Park, Bardon
Hill, Coalville, Leicestershire, LE67 1UB, UK.



NRS HEALTHCARE 4C Altona Road, Blaris
Industrial Estate, Lisburn, Co Antrim,
Northern Ireland BT27 5QB

Tel: Customer Services: 0345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.

Please visit one of our websites to view our full range of products:

www.healthcarepro.co.uk or www.completecareshop.co.uk