

Introduction

Congratulations on your purchase of the Cefndy Healthcare T220 Tilt in Space Toileting/Showering Commode Chair. Please ensure you read and understand these instructions in full before construction and use. Please retain these for future reference.

This chair is designed for those with more complex support requirements. It provides comfort, safety and easy access for washing and toileting and can be reclined up to 30 degrees.

The neck support is fully adjustable and can be positioned at any angle for maximum user comfort. A lap strap is included for added safety.

The chair features footplates that can be independently inclined to suit the user's requirements. The frame is manufactured from powder coated aluminium with PU seat and back for patient comfort. The PU seat can be removed for ease of cleaning. Also, the potty holder is integrated into the PU seat. Both arms fold up to allow lateral transfers.



Specifications

Overall Height with Neck Brace Fully Extended	1360mm
Overall Width	560mm
Overall Depth with Footplates	890mm
Seat Height	535mm
Seat Depth	420mm
Seat to top of Back	485mm
Orifice Depth & Width	266 x 190mm
Potty Diameter	300mm
Width Between Arms	460mm
Tilt Angle	30°
Belt Length	1115mm
Maximum User Weight	159kg / 25st

Pack Contents

Inside the box you should find:

- Tilt in Space Toileting / Showering Chair
- Commode Potty
- Easy Clean PU Back
- Easy Clean PU Seat
- Slot in Plug
- Lap Belt
- User Instructions (this leaflet)

Assembly Instructions

Please note:

- When unpacking carefully examine the product to ensure it is complete. See contents list above.
- Before you begin assembly check for any damage. If damaged please contact your supplier and refrain from using the product.
- **Do not attempt to use if the product is damaged.**

Assembly:

- Put the seat belt on the backrest tube and assemble the backrest tube to the frame; using M8 knob to fix backrest on the frame; put the brake wire to C clip.
- Take out the castors from the carton and assemble on the frame. Front and rear castors are all swivel castor with brake, using E clip to fix the position.
- Take the footplates out and put it on the frame, sway it inside.
- The footplates can be adjusted to an angle if necessary. Using Allen wrench to loosen the screw, adjust the angle and lock the screw.
- Place the PU headrest to fixed set on the push bar. The headrest can be adjusted to a different angle; the knob is controlled by the distance of the headrest.

Correct Use

- The armrest can be swayed up and down by the user.
- Footplates can be folded by pressing the lever control to make the footplate go out. To return the footplate position, push the footplate from outside to inside.
- To use the commode, take off the seat cover, pull the commode from the back, remove the commode lid and place the commode back to use.
- The seat angle can be reclined by pressing the lever control on the handle and pushing down. To return the chair to the seating position, press the lever control.
- Please empty the potty at regular intervals and be aware of its maximum capacity.

Health & Safety

Customer/User Please Note: Do not attempt to move the product if the castors are locked. This is because the castors are on an expansion bolt, not a pin clip – with the expanding bolt they could become undone if turned whilst the brake is on.

Cleaning

- Use mild detergent with warm water to wipe down framework on at least a weekly basis more frequently if required.
- Do not use an abrasive cleaner as this may damage the frame and invalidate your warranty.
- It is recommended that the potty is thoroughly cleaned after every use, using an appropriate cleaning agent.
- Keep the painted surface clean from oil.

Use of power/jet washing & industrial washing machines:

- If either of the above is used to clean equipment care must be taken since high-pressure water can be penetrative.
- Care should be taken that frames are not placed too close to high-pressure water jets.
- After cleaning ensure any residual water is removed from the framework as it could cause permanent rusting of the frame (leave frames open in a warm environment for several hours to ensure all internal moisture evaporates).
- We cannot offer advice on industrial washing machines with regards to specific machine settings, as these will be dependent on the manufacturer's instructions.

Maintenance Requirements

- The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking paint. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.
- Check the product every three months: clean the castors, tighten the screws and check the brake functions on the castors.

Warning

- This product is intended for users of a maximum of 159kg/25 stone in weight.
- We recommend a full risk assessment should be carried out before use.
- Under no circumstance should the commode be moved whilst occupied.
- It is advised that the frame is checked periodically for signs of wear and tear. Any problems please report to your supplier immediately.
- Please make sure E clip goes through tubing and the castor is well fixed to prevent the E clip coming off and causing the castors to slant.

Warranty

This product carries a 12 month warranty from the date of purchase against manufacturing defects. Misuse or damage is not covered. This warranty does not affect your statutory rights. As manufacturers we cannot specify expected product life as this is based on the level of usage, however we can advise that this product has been designed, manufactured and independently tested in the UK (including cyclic testing to represent real life usage).

Disposal Instructions

- Appropriate hygiene standards should be applied before disposal to reduce the risk of infection.
- Materials used are essentially non-hazardous and can be disposed of appropriately.

Contact Details

We have attempted to cover most eventualities, however if you have any specific queries, please contact:

Cefndy Healthcare
Cefndy Road
Rhyl
Denbighshire
LL18 2HG
Tel:01824 706005

E-mail: cefndy.sales@denbighshire.gov.uk