

aquila[®]

bath lift

Pelvic harness
fitting instructions

Each harness consists of one fixed and one adjustable section, which connect around the user with a buckle (see fig. 1). These sections have colour-coded tags located near the white strap fixings to ensure they match together correctly (see fig. 2).

Child pelvic harness (AB229):

- 1 x fixed section with red tag
- 1 x adjustable section with green and red tags

Adult pelvic harness (AB230):

- 1 x fixed section with yellow tag
- 1 x adjustable section with yellow and blue tags

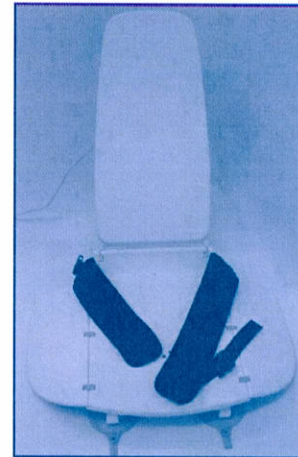


Figure 1

1. Attaching the pelvic harness

For easy reach and length adjustment, we recommend the harness's buckle is always located on the side closest to the carer.

- Insert the white strap fixing of the harness's adjustable half (i.e. with 2 coloured tags) into the seat eyelet on the furthest side from the carer (see fig. 3).
- To lock, rotate the strap fixing until the harness rests on the seat.
- Repeat with the fixed section on the other side of the seat.
- Position the fixed section over the pelvis and bring the adjustable section tightly over it (see fig. 4).
- Connect the buckle and pull the strap to tighten the harness as required (see fig. 5).



Figure 2

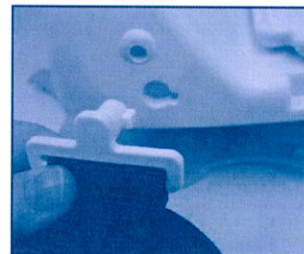


Figure 3

2. Cleaning the pelvic harness

Remove the harness sections by rotating the white strap fixings free of the eyelets in the seat.

They can now be washed in an automatic washing machine at 40°C. Do not tumble dry.



Figure 4

3. Material specification

Polyester laminated neoprene with nylon strap fixings.



Figure 5

Mountway Limited

Tafarnaubach Ind. Est. Tredegar NP22 3AA UK
tel +44 (0)1495 723300 fax +44 (0)1495 723360
e-mail mail@mountway.co.uk
www.mountway.com

