

# User Instructions

## Economy Shower Chair Range

**Maximum  
User  
Weight:**



### General Safety Warning

**Please ensure these instructions are fully read and understood before installation/ use. Retain these instructions in a safe place for future reference.**

### 1.0 Intended Use

This range of shower chairs is intended to be used in a bathroom or shower, to provide a stable seated surface for clients with reduced mobility who are unable to stand during showering.

### 2.0 Safety Precautions

- Do not exceed the maximum user weight.
- Ensure seat is correctly fitted and secure.
- Inspect product at regular intervals see section 5.2
- Do not use if feet or castors are damaged.
- Ensure all castors are locked when the seat is in the desired position and if leaving the client temporarily.

### 3.0 Unpacking

Check all parts carefully before assembling the product. Do not assemble if any parts are damaged.

### 4.0 Assembly and Height Adjustment

- Where the seat is supplied separately, fit to the frame by first clipping it onto the rear of the two closely spaced cross bars, and then clipping to the front cross bar.
- On height adjustable models, adjust the legs by sliding to the desired height and secure with the fixing e-clips.
- Set all legs to the same height.
- Check the seat is level and the product is stable before use



**L99387**  
**Fixed Height**



**L99399**  
**Adjustable Height**



**L99405**  
**Mobile Fixed Height**



**M00754**  
**Mobile Adj. Height**

### 5.0 Product Care

#### 5.1 Cleaning Instructions

- Clean regularly, use a general purpose, neutral detergent and warm water
- Please check the suitability of the product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.
- Do not use any form of solvent as this will damage the surface and weaken the product and again invalidate any warranty.
- To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water
- These seats may be autoclaved on cleaning cycles up to 73°C
- Dry the seat thoroughly before use.

## Product Care - Continued

### 5.2 Maintenance

- Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the ferrules and castors
- Check that all fixing clips are inserted correctly and there is no excessive movement.
- Check for general wear or damage.
- If you see or suspect a fault do not use – contact your dealer immediately

### 6.0 Product Guarantee

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover, abuse, misuse, interference or tampering with any part of the product. Any guarantee issued by NRS Healthcare does not affect your statutory rights.

**From the NRS Healthcare Range, you may also like:**

Nuvo Adjustable Bath Step  
Code No: M11077



Spa Curved Grab Rail  
M53527 Length: 355mm (14")  
M53539 Length: 480mm (19")  
M85553 Length: 620mm (24 1/2")



Combi Grabber  
N72358 Length: 660mm (26")  
N72369 Length: 813mm (32")



### NRS Healthcare

Sherwood House, Cartwright Way,  
Forest Business Park, Bardon Hill,  
Coalville, Leicestershire.  
LE67 1UB UK

Tel. Main Office: 0330 100 00 98  
Tel. Customer Services: 0345 121 8111  
Email: [customerservice@nrshealthcare.co.uk](mailto:customerservice@nrshealthcare.co.uk)



**NRS Healthcare** is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:  
**[www.nrshealthcare.co.uk](http://www.nrshealthcare.co.uk)**