

User Instructions Cefndy Drop Down Grab Rail

G29 – G30 – G290 – G300

Introduction

Congratulations on the purchase of your new Drop down Rail from Cefndy Healthcare. Please ensure you read and understand these instructions in full before use. Please retain these for future reference.

The range offers a stylish but strong wall fixing rail, designed to aid toileting. It has an easy up/down mechanism.

The range is available in several versions and includes rails with adjustable support legs, trombone style and red rails.



Product Specifications

| | G29 | G30 | G290 | G300 |
|-------------------------------|--------------------------|---------------------------|--------------------------|---------------------------|
| Overall Length | 700mm 27½" | 700mm 27½" | 720mm 28¼" | 720mm 28¼" |
| Max Distance from Wall | 145mm 5¾" | 100mm 4" | 145mm 5¾" | 145mm 5¾" |
| Mounting Bracket | 260 x 100mm 10¼" x 4" | 260 x 100mm 10¼" x 4" | 260 x 100mm 10¼" x 4" | 260 x 100mm 10¼" x 4" |
| Adjustable Leg | --- | 690 – 780mm 27" – 30¾" | --- | 740 – 880mm 29" – 34¾" |
| Maximum User Weight | 127kg 20st | 190kg 30st | 115kg 18st | 115kg 18st |

Pack Contents

- Drop Down Rail
- User Instructions

User Instructions Cefndy Drop Down Grab Rail

G29 – G30 – G290 – G300

Assembly Instructions

Please note:

- When unpacking carefully examine the product to ensure it is complete.
- Before use check for any damage. If damaged please contact your supplier and refrain from using the product.
- **Do not attempt to use if the product is damaged.**
- Place the Grab Rail in the required position at the correct height and orientation.
- Mark out all the hole positions on the wall.
- For walls, drill and plug wall with a suitable sized drill and plastic insert. If tiles are to be drilled, particular care should be taken not to slip whilst drilling and not to damage the tiles. If in doubt, seek advice from professionals.
- Fix the rail to the wall with screws of the appropriate type ensuring all the screw holes are utilised.

Correct Use

- Grab Rails should only be fixed to objects of solid construction and careful consideration should be given to choosing the appropriate fixing method – **if in doubt seek advice from professionals.**

For G29 / G30 / G30R

- To lower the grab rail from its stored (upright) position, hold the large grey end cap on the oval tubing and pull it vertically upwards to release the safety catch. The rail should then be slowly lowered into its working position.
- To store the rail after use, hold the grey end cap and push the rail to the vertical - it will then lock itself into this position.

For G290 / G300

- To lower the rails from its stored (upright) position, take hold of the rail and slowly lower it to its working position.
- To store the rail after use, hold the rail and push into vertical position until it clips into place.

User Instructions Cefndy Drop Down Grab Rail

G29 – G30 – G290 – G300

Cleaning

- Use mild detergent with warm water to wipe down framework on at least a weekly basis more frequently if required.
- Do not use an abrasive cleaner as this may damage the rail and invalidate your warranty.

Use of power/jet washing & industrial washing machines

- If either of the above is used to clean equipment care must be taken since high-pressure water can be penetrative.
- Care should be taken that rails are not placed too close to high-pressure water jets.
- After cleaning ensure any residual water is removed from the inside of the rail as it could cause permanent rusting of the rail (leave rails in a warm environment for several hours to ensure all moisture evaporates).
- We cannot offer advice on industrial washing machine with regards to specific settings as these will be dependent on the machines manufacturer's instructions.

Maintenance Requirements

- The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking paint. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.

Warning

- This product is intended for users of a maximum of 127/20st (G29) or 190kg/30st (G30) in weight.
- It is recommended that a user specific risk assessment is undertaken before use. This should include consideration for the equipment's intended use, the capacity of the user to safely use the equipment, the environment in which the equipment is intended to be used and all other relevant considerations.
- Grab Rails should only be fixed to walls of solid construction and careful consideration should be given to the method of fixing - **If in doubt seek advice from professionals.**

User Instructions Cefndy Drop Down Grab Rail

G29 – G30 – G290 – G300

Warranty

This product carries a 12 month warranty from the date of purchase against manufacturing defects. Misuse or damage is not covered. This warranty does not affect your statutory rights. As manufacturers we cannot specify expected product life as this is based on the level of usage.

Disposal Instructions

- Appropriate hygiene standards should be applied before disposal to minimise the risk of infection.
- Materials used are essentially non-hazardous and can be disposed of appropriately consideration should be given to local re-cycling arrangements.

Contact Details

We have attempted to cover most eventualities, however if you have any specific queries, please contact:

Cefndy Healthcare
Cefndy Road
Rhyl
Denbighshire
LL18 2HG
Tel: 0044 1745 343877
E-mail: cefndy.sales@denbighshire.gov.uk