

USER INSTRUCTIONS

NRS HEIGHT ADJUSTABLE TOILET FRAME & SEAT

P11942

NRS Height Adjustable Toilet Seat and Frame

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

Maximum User Weight



1. INTENDED USE

To assist those who have difficulty in lowering onto or rising from a standard toilet. It should be used over a toilet using the splash guard chute supplied.



2. SAFETY

Warning: The maximum user weight must not be more than 190 kg (30 st).

3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

4. ASSEMBLY/INSTALLATION

Fit the splash guard chute by raising the toilet lid and seat and placing onto the wire side brackets

5. IN USE

Adjust to the required height by removing each 'e-clip' in turn.

Slide each leg up or down to the required height. Align the holes and reinsert the 'E-clip'.

Ensure that all four legs are set at the same height to prevent rocking. The device may then be placed over the toilet and is ready for use.

6. MAINTENANCE

Check the feet monthly for signs of wear.

Check 'E-clips' regularly for signs of wear.

7. CLEANING INSTRUCTIONS

Clean regularly using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as these could damage surfaces and invalidate any warranty.

To disinfect, wipe with a dilute bleach solution and rinse thoroughly.

Allow the product to dry prior to use/storage.

8. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

9. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: +44 (0)345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk