

## Product Technical Data Sheet 088 Atlas Advance Knee Pad Covers

### Product Range

P18808 Knee Pad Covers

### Product Description & Intended Purpose

The transfer aid knee pad covers offer additional comfort to the user of the Atlas Advance Transfer Aid. The extra cushioning provides greater protection for users with frail or thin skin, helping to increase pressure relief.

### Product Features

- Padded cushion for comfort.
- Wipe clean waterproof cover.
- Shaped design moulds over knee pad.




### Technical Specification

**Length:** A - 195mm (7½")

B - 185mm (7¼")

**Width:** C - 155mm (6")

**Weight:** 0.1kg

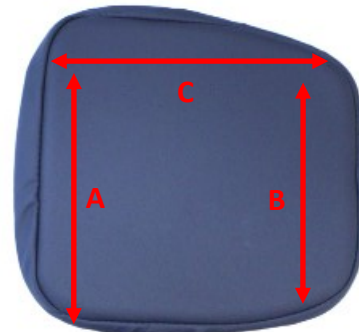

### Safety Precautions

- Ensure the covers are correctly fitted over the knee pads prior to use.

### Installation Information: *(Note: for detailed information refer to User Instructions)*

- Position the knee pad cover over the knee pad such that the seam is on the outside edge.
- Pull the top elastic strap over the top of the knee pad.
- Stretch the cover to fold over the top of the knee pad.
- Pull the bottom elastic strap up around the bottom of the knee pad.
- Stretch the cover to fit tightly over the bottom of the knee pad.
- Repeat for the second knee pad and cover.

## PTDS088

### Atlas Advance Knee Pad Covers

#### Cleaning

- Remove the covers from the knee pads prior to washing. For spot cleans use a damp cloth.
- Machine wash at a maximum temperature of 40°C using a mild detergent. Do not use bleach. Do not steam clean. Do not iron.
- Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will permanently damage the cover and invalidate any warranty.
- Allow covers to air dry thoroughly before refitting to knee pads. Do not tumble dry.

#### Care & Maintenance

- Periodically inspect the product for signs of wear. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

#### Accessories / Related Products

- P11437 - Atlas Advance Transfer Aid

#### FAQ's

#### Product Supplied By:

**NRS Healthcare** Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire. LE67 1UB UK