

User Instructions

Bathroom Safety Rail L97780

General Safety Warning

Please ensure these instructions are fully read and understood before installation & use. Retain these instructions in a safe place for future reference.

Intended Use

To help steady the user while transferring into or out of the bath. It is not intended to be weight bearing and should not be used by the user to pull up from a sitting position.

Installation Instructions

WARNING: this product may not fit properly on certain unusual or sculptured baths.

Inspect for fit and stability before fitting/use.

USE EXTREME CARE when installing the Bathroom Safety Rail onto fibreglass or plastic baths. DO NOT overtighten, as clamp may crack the bath.

1. Clean bath wall area where Bathroom Safety Rail is to be applied thoroughly. All contact surfaces must be dry and free of soap residue to ensure secure attachment.
2. Loosen adjustment hand-wheel by rotating anti-clockwise until the platform opening is wide enough to fit around top of bath wall.
3. Place Bathroom Safety Rail onto bath, with hand-wheel facing outside the bath and tighten snugly by turning the hand-wheel clockwise.
4. Test the Safety Rail is secure by pulling backwards, forwards and side to side. If the unit moves, reposition and refit until secure.

NRS Healthcare

Sherwood House, Cartwright Way, Forest Business Park,
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK
Tel. Customer Services: +44 (0)345 121 8111
Email: customerservices@nrshealthcare.co.uk



Safety Precautions

DO NOT use Bathroom Safety Rail to pull up from a sitting position.

Periodically check the stability and rigidity of the Bathroom Safety Rail.

Cleaning Instructions

Clean regularly, wiping down with a mild detergent or disinfectant solution.

Do not use an abrasive detergent, cleaner or cloth to clean or dry this product.

Product Guarantee

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship. This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS Healthcare is a leading supplier of community equipment and daily living aids.

Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk

UI/L97780/08/23/Iss6