



Instructions

This TreadNought Floor Sensor Pad is designed to be connected directly to a Nurse Call System to activate the Nurse Call when the person in your care steps onto the TreadNought Floor Sensor. This TreadNought Floor Sensor has been supplied with the necessary components to allow your floor sensor to be connected to a standard Mono Maker or Stereo Maker type nurse call system. Additional parts may be required if you do not have a standard Mono Maker or Stereo Maker Nurse Call System.



WARNINGS AND CAUTIONS



Read these instructions and the warnings and cautions that follow carefully before using the TreadNought floor mat. Failure to read and follow these instructions and warnings and cautions may result in injury or death to a person in your care or system damage or failure.

- This product is an aid only and may not be suitable for all patients. It is not a substitute for staff vigilance or visual monitoring.
- Test before each use.
- Discard and replace the TreadNought mat one year from the date of first use.
- Apply TreadNought anti-slip mesh material (sold separately) to the underside of the TreadNought floor mat if the floor surface is slippery and the mat slides when stepped on.
- Position the TreadNought cord so that it is not a tripping hazard for the patient, staff, or visitors.
- Clean the TreadNought mat by wiping it off with a clean rag and vinyl-tolerant cleaning solution. Do not immerse.
- Store flat. Do not bend, fold, or roll the TreadNought floor mat.

To Set Up the TreadNought Floor Sensor Mat

- 1) Use a permanent marking pen to record the date of first use in the space provided on the TreadNought floor mat label.
- 2) Place the TreadNought floor mat in the desired monitoring location with the smooth side facing down.
- 3) Position the TreadNought cord so that it is not a tripping hazard for the patient, staff, or visitors.
- 4) Check to see if the mat slides on the floor. If there is any slipping or sliding, use double-sided adhesive tape to apply TreadNought anti-slip mesh material (sold separately) to the underside of the mat.
- 5) Test again for slipping. If the mat continues to slide in that location, move it to another location.
- 6) To identify whether your nurse call system requires a Mono or Stereo Jack, please refer to the client call cord (pear push lead). The connector type should be a silver jack plug and will have either 1 black ring (Mono) or two black rings (Stereo). If your Nurse Call system requires a Mono Jack, the lead from the TreadNought Floor Sensor can be connected directly into the socket on your Nurse Call system. For Nurse Call Systems that require a stereo Jack, connect the Mono to Stereo Jack adaptor (NCPLUGSTEREO) on to the end of the Mono jack lead of the TreadNought Floor Sensor and then plug into your Nurse Call System
- 7) Insert the TreadNought plug securely into the monitor or call cord receptacle of your nurse call system.
- 8) If you only have one available connection point on your nurse call system but need to have both the TreadNought Floor Sensor and clients call cord available you will need to use the T-Connector supplied. First follow step 6 to determine whether your Nurse Call System requires a Mono or Stereo connection and adapt the jack plug on the TreadNought Floor Sensor if necessary. Plug the T-Connector in to your Nurse Call System, plug the cord from the TreadNought Floor Sensor into one of the two sockets of the T-Connector and then plug the clients call cord into the other available socket.

-OVER-

Using the TreadNought Floor Sensor Mat



WARNING: The TreadNought floor mat is an aid only and may not be suitable for all patients. This product is not a substitute for staff vigilance or visual monitoring.

Always Test Before Each Use

- 1) Step on the TreadNought floor mat.
- 2) The alert will sound on the monitor or nurse call system.
- 3) Reset the alert.

If the Alert Does Not Sound

- 1) Check your connections.
- 2) Test again.
- 3) If the alert still does not sound, do not use the mat.
- 4) Contact your sales representative for further assistance.

To Clean the TreadNought Floor Mat

- 1) Wipe the mat with a clean rag and vinyl-tolerant cleaning solution.
- 2) Do not immerse the mat.

Specifications

- Minimum weight to trigger the alert: 2 kg
- Size: 600 mm X 900 mm X 7mm
- Cable length: 2.5m
- Product weight: 1.5 kg



Turun UK, Ltd.

Eden House 59 Fulham High Street

London, SW6 3JJ UK

Phone: 020 7731 0132

Free Phone: 0500 500 667