

# Tray Range

Description	Contents
Stay Tray	Tray, Bean Bag and Anti Slip Surface
Freehand Tray	Tray with Carry Handle and Silicon Mat
Variant Freehand Tray	Tray with a Carry Handle

## Stay Tray

A sturdy light weight plastic tray with a bean bag to allow the tray to be used as a lap tray providing a stable surface. Incorporates an anti slip surface on the face and base of the tray that prevents items sliding around. The Bean Bag filled with polystyrene beads is secured to the base of the tray with hook-and-loop fastening, this allows the bag to be detached from the tray. Large enough for a full meal.

## Freehand Tray

A strong plastic tray with a unique handle that allows you to carry the tray and its contents with only one hand, allowing you to open doors, climb stairs and support yourself safely whilst carrying items. The handle folds flat inside the inner rim of the tray and locks into place in the upright position, ensuring your hands do not become trapped when you put the tray down. The handle can easily be removed for cleaning or use as a standard tray. Large enough for a full meal and has a lower edge on one side allowing your arm to rest on the tray. Supplied with a silicon non-slip heat-resistant mat that prevents items sliding around, even when the tray is at an angle (mat colour may vary).

## Variant Tray

A tray that provides all the features of the Freehand Tray, without the Mat.

## Cleaning and Care

1. Wipe the tray over with a solution of washing up liquid and water, dry immediately. Not suitable for a dishwasher.
2. The non-slip mat will only work if it is kept free of dirt and dust and should not be used when wet as this will reduce the non-slip resistant qualities.
3. Place mat on tray shiny side down.
4. Should be washed regularly with any non-abrasive household cleaner or mild detergent.
5. Do not put very hot plates on to the mat.