



USER GUIDE – NON-SLIP REELS

Dycem has a range of Non-Slip Rolls which are available in different sizes and colours, all can be cut to size and shape to fit discreetly under items in the clinic or home. The reels grip on both sides without adhesive.

Available in two widths:

20cm or 40cm

Various Lengths:

1m, 2m, 5m, 9m and 15m

How to use Dycem Non-Slip Reels:

- Simply unroll the material from the reel
- Cut to the required size and shape
- The product is then ready to use.

Typical uses:


- Under plates and mixing bowls for independent eating and dining
- On top of trays and tilt tables
- Underneath pillows
- On armrests and seats to prevent sliding.





Dycem®

Please let us know how you use your Dycem product, simply email yourideas@dycem.com or tag us on **Twitter** to be entered into our monthly draw!

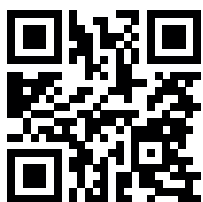

Dycem Ltd

Ashley Trading Estate,
Bristol BS2 9BB, United Kingdom

Tel: +44 (0) 117 955 9921
Fax: +44 (0) 117 954 1194
nonslip@dycem.com

Dycem Corporation

33 Appian Way, Smithfield,
RI 02917, USA

Tel: 1-800 458 0060
Fax: 401-738-4420
nonslip@dycem.com

© 2020 Dycem Ltd. All rights reserved

How to clean:

When the reel material becomes dusty simply restore the non-slip properties by washing the reel in warm soapy water and allowing to dry.

Alcohol wipes can also be used to clean the products.

Useful to know:

Dycem contains in-built antimicrobial technology which provides effective 24/7 product protection against microbes that may cause degradation, stains or odours.

Please Note:

- Dycem is not effective when wet so the products should not be used in the bathroom.
- Please do not subject to temperatures over 50°C (122°F).
- Dycem products should not be placed in a microwave or autoclave.



Making life easier

www.dycem-ns.com