

homecraft White Line Swivelling Bath Seat

Product Code	09 147 7769
---------------------	--------------------

The following manual provides descriptions relating to the assembly, operation and care of the Homecraft White Line Swivelling Bath Seat.

Please consult your doctor prior to using the product to confirm that this is the appropriate aid for you. If you have any other questions about this bath seat, please contact your supplier.

Use in accordance with the requirements

The Swivelling Bath Seat offers individuals with restricted mobility but still sufficient sitting capacity the option to carry out daily personal hygiene in the bath tub.

The bath seat makes it possible to use the bath tub while sitting approximately at the height of the bath tub rim. The turning seat makes getting on and off easy.

Possible conditions or results of injury are e.g.:

- Impaired function of the upper body movement or position (e.g. neuromuscular diseases or impaired balance) with reduced or lacking function of the lower and / or upper extremities (e.g. amputation, paralysis, joint disease).

The Swivelling Bath Seat is only intended for use in connection with standard bath tubs.

The Swivelling Bath Seat must only be used in bath tubs in which a secure position of the seat is guaranteed. The Swivelling Bath Seat must only be used when fully assembled.

Safety – General notes

Please read the assembly instructions for the Swivelling Bath Seat carefully before starting the assembly!

Do not continue to use the Swivelling Bath Seat if defects are detected during use.

Please take care that:

- The instructions are read by all individuals who will use the Swivelling Bath Seat.
- All individuals using the Swivelling Bath Seat have access to the instructions at any time.

Damages resulting from a failure of observing the instructions are excluded from any warranty.



Restrictions of use!

- Patients weighing more than 130kg must not use the Swivelling Bath Seat.



Maximum
user weight

130
kg **20**
st



Accident risk!

- Only use the Swivelling Bath Seat for its intended use. They are not replacements for ladders and are not exercise equipment or toys.
- The Swivelling Bath Seat is not intended for use during examinations or treatments.
- Use the Swivelling Bath Seat only in technically perfect condition.
- Use only spare parts and accessories released by Performance Health International Ltd.
- Do not carry out structural changes which are not approved by the manufacturer.
- The Swivelling Bath Seat must not be used in environments in which flammable gases or vapours (e.g. from anaesthetics) can be expected.

Notes on the assembly of the Swivelling Bath Seat



Risk of injury due to wrong assembly!

- Check the perfect position and firm support of all parts after the assembly.




Risk of injury!

- Use only tools appropriate for the work.
- Carry out the work only if you are familiar with using the tools employed.

These user instructions are available in large print, please contact Performance Health Customer Services for a copy.
Please retain these instructions for future reference.

Manufactured for:

 **Performance Health International Ltd.**
Nunn Brook Road, Huthwaite,
Sutton-in-Ashfield, Nottinghamshire,
NG17 2HU, UK.

UK:

Tel: 03448 730 035
Fax: 03448 730 100
www.performancehealth.co.uk

International:

Tel: +44 1623 448 706
Fax: +44 1623 448 784

Performance Health Supply, Inc.

W68 N158 Evergreen Blvd.
Cedarburg, WI 53012
USA
www.performancehealth.com

Performance Health ANZ

Unit 3, 3 Basalt Road,
Pemulwuy NSW, Australia 2145.
Tel: 1300 473 422
Fax: 1300 766 473

Performance Health France

13 rue André Pingat
CS 10045
51724 Reims Cedex
France
Tel: 00(33) 03 10 00 79 30
Fax: 00(33) 03 54 70 04 34
www.performancehealth.fr

 **PERFORMANCE
HEALTH**

Homecraft™ is a trademark of Performance Health International Ltd.

Contents

The Swivelling Bath Seat bath seat is packed in carton.

Use the following list to check the contents of the package before starting the assembly.

Please contact your supplier if defects are apparent or if parts are missing.

Contents:

- 1 × (1) Seat
- 1 × (2) Frame

Preparations


Observe the safety notes

NOTE

- Put seat (1) from above onto the pintail (3) of the frame (2) up to the stop.
- Tighten the locking screw (4).

NOTE



- Tightening too much results in the destruction of the threaded insert.

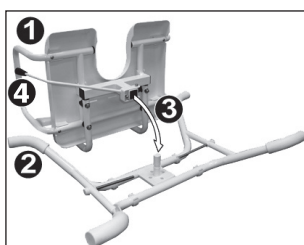

Attaching the bath seat to the tub

Only use the bath seat in bath tubs for which the secure support of the rubber caps (1) is guaranteed.

At least 2/3 of the width (B) of the straight section of the rubber caps has to be supported by the bath tub rim (2).

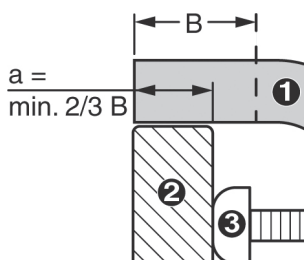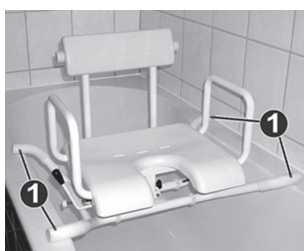

a (Support width) = min. 2/3 B


- Place bath seat on the bath tub rim.
- Align the centre of the bath seat and screw both locking screw (3) out to the bath tub rim.

NOTE

The locking screws (3) prevent slipping of the bath seat.

Accident risk!

- **Make sure before use that the bath seat rests securely on the bath tub rim and that the locking screws fit the tub edge closely.**

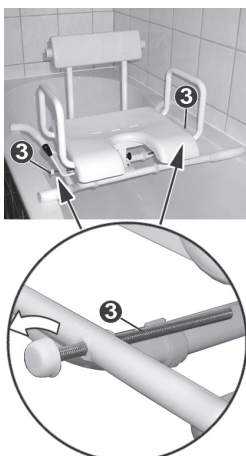



Turning the bath seat

⚠ Risk of crushing and trapping!

High risk of crushing and trapping exists for all joints and in the turning range of the bath seat.

The bath seat can be stopped in 4 positions, turned by 90°, to make getting on and off easier.

- Pull locking lever (1) up and turn the bath seat into the desired position.
- Release the locking lever to lock the bath seat into place. It locks automatically into the relevant turning position.

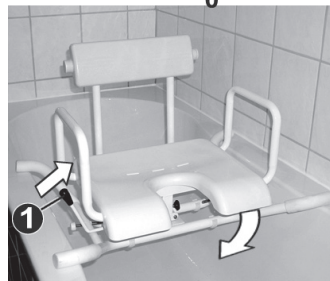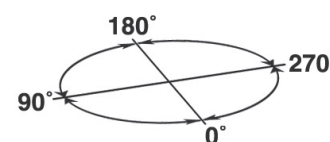

⚠ Accident risk due to unlocked turning adjustment!

- After turning the bath seat, swivel it slightly back and forth to ensure that it is firmly locked in place.

Cleaning

- Small stains and dust are best removed with moist wipes.
- Larger stains can be removed by non-scouring, mild cleaning agents.
- Do not use tools with sharp edges (knives, metal scrapers) or aggressive solvents for cleaning.

Maintenance

The Swivelling Bath Seat is maintenance-free, however, should be checked on a monthly basis.

Carry out the following tests:

- Visual check of all parts for deformation, damage, wear and tear and corrosion.
- Check all connections (nuts, screws etc.) for completeness, function and perfect fit.

The seat must not be used further if defaults are detected during use.

Spare parts

Spare parts can be obtained from your supplier. Only original spare parts may be used.

Disposal

The Swivelling Bath Seat consists of metal and plastic parts. Disposal of the individual materials must be carried out according to the regional environmental and disposal provisions.

Warranty

Performance Health International Ltd offers a 24 months warranty on perfect quality within the framework of its general terms and conditions.